

# TARGET™ HCG

## Serum Control Set

### Catalog #30-0104

#### **INTENDED USE**

Target HCG Serum Controls are used for the assessment of HCG levels with qualitative HCG test kits

#### **REAGENTS**

HCG Serum Controls: buffered human serum with 0.1 % sodium azide as preservative

**Precaution:** For IN VITRO diagnostic use. Reagents containing sodium azide may combine with copper or lead plumbing to form highly explosive metal azides. Dispose of reagents by flushing with large amounts of water to prevent build up.

#### **Storage and Stability**

Target Serum Controls are ready for use. Store at 2 – 8 degrees C. Expiration date provided on vial labels. Avoid microbial contamination of reagents.

#### **Material Provided**

1 X 5 ml Serum HCG Control: Negative ( $\leq 5\text{Miu/ml}$ )

1 X 5 ml Serum HCG Control: Positive ( $400 \pm 25\text{Miu/ml}$ )

#### **PROCEDURE**

Refer to procedure section in package insert supplied with your HCG test kits.

#### **RESULTS**

Results are recorded as positive if they produce a visible reaction in the “Test” area Of the cassette.

Specimens that do not produce a reaction in the “Test” area should be recorded as Negative.

The absence of a reaction in the “Control” area after the test procedure indicates that the correct test procedure was not followed or reagents were not reactive and that the test should be repeated.

MYERS-STEVENSON GROUP

7137 Telegraph Rd.

Montebello, CA 90640

Tel.: (888) 313-9191

Tel.: (323) 721-8552

Fax: (323) 721-8554

[www.myersstevensgroup.com](http://www.myersstevensgroup.com)

March 2007

D300104-1

